


# MetaClip-P & Appliers

## Polymer Ligating Clips & Appliers




### – MetaClip-P Features

- Non slip type
- Clear Locking sound
- Biocompatible Polymer
- Non-absorbable Polymer
- Flexible Hinge
- High tensile strength

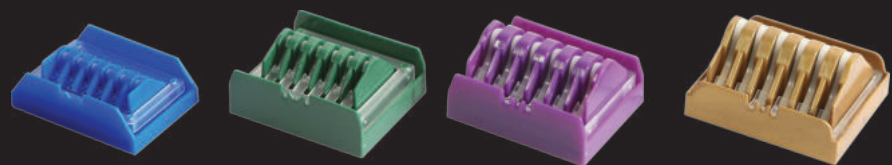

### – Clip Specification

Vessels / Tissues bundle size range

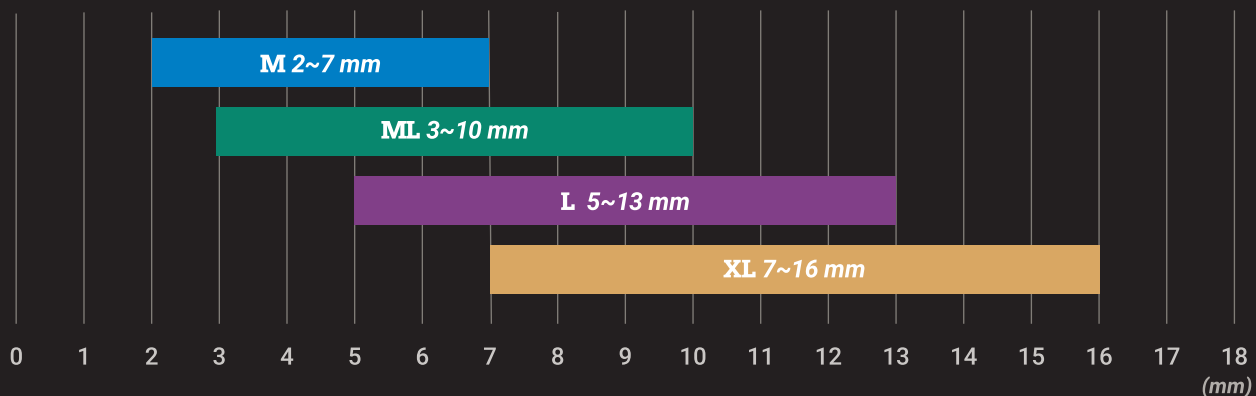


Clip	Ref No.	Clip / Cartridge	Cartridges / Box	Clips / Box
M	PCMM01A	6	14	84
ML	PCML02A	6	14	84
L	PCLL03A	6	14	84
XL	PCXL04A	6	14	84

# CLIP

## Non-Absorbable Polymer

is a biocompatible polymer that does not interfere with CT, MRI or X-ray.

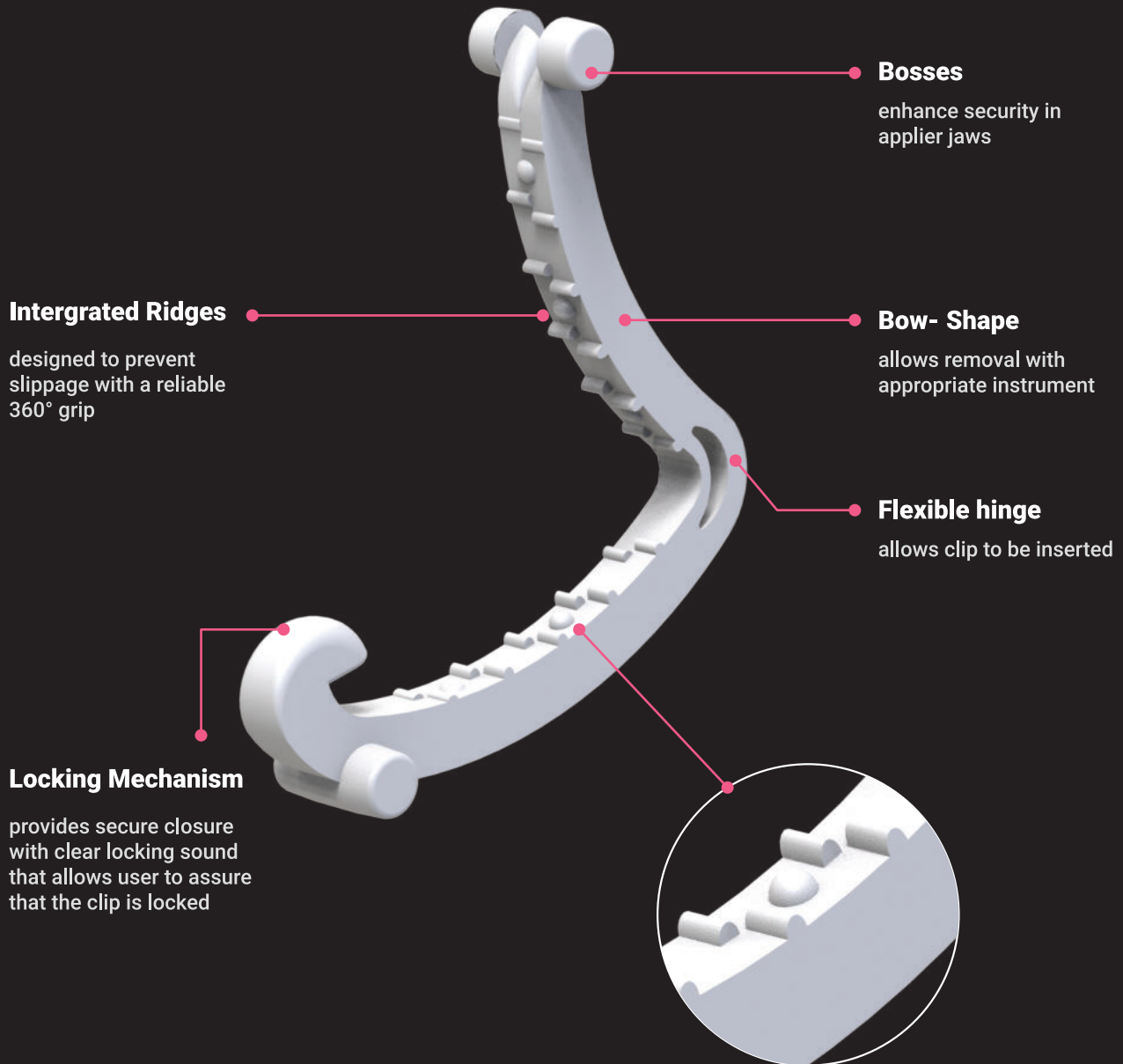

### The teeth is designed to prevent slippage

The circular projection inside the clip prevents slippage

# Appliers

## Proper Jaw Angle

for better loading it is necessary to position the applier jaw perpendicular to the cartridge body



## Appliers Features

- Non - detachable design for easier maintenance and durability
- Made of high quality stainless steel
- Color coded size matching cartridge
- Compatible with corresponding sizes of Meta Clip-P polymer



## Applier Sterilization

### Sterilization Conditions

Wrap with a disinfection Cloth.

### Instrument State

Pre-vacuum

APPLIER	TEMPERATURE	EXPOSURE TIME	DRYING TIME
PREMIUM APPLIER	132° C	4 minutes	30 minutes
STANDARD APPLIER	134-135° C	3 minutes	30 minutes

◦ **OPEN APPLIER (PREMIUM)**




Clip	Model No	Length
ML	OPAPMLA	210mm
L	OPAPLLA	210mm
XL	OPAPXLA	210mm

◦ **OPEN APPLIER (STANDARD)**




Clip	Model No	Length
M	OMAPMMA	280mm
ML	OMAPMLA	280mm
L	OMAPLLA	280mm
XL	OMAPXLA	280mm

◦ **ENDO APPLIER (PREMIUM)**

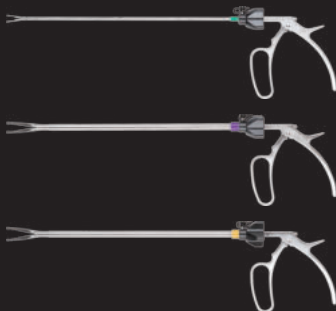


Clip	Model No	Shaft Length	Shaft Diameter
ML	ENAPMLB	330mm	5mm
L	ENAPLLB	330mm	10mm
XL	ENAPXLB	330mm	10mm

◦ **ENDO APPLIER (STANDARD)**

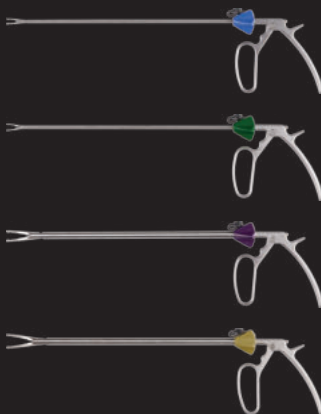

Clip	Model No	Shaft Length	Shaft Diameter
M	EMAPMMA	330mm	5mm
ML	EMAPMLA	330mm	5mm
L	EMAPLLA	330mm	10mm
XL	EMAPXLA	330mm	10mm

## ○ OPEN REMOVER

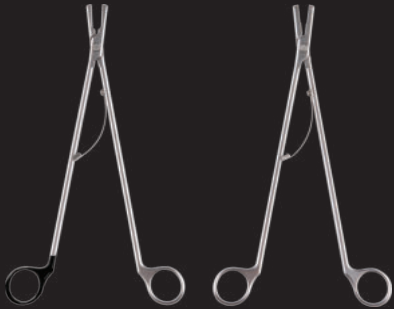

Clip	Model No	Length
M/ML	ORAPMLA	280mm
L/XL	ORAPXLA	280mm

## ○ ENDO REMOVER



Clip	Model No	Shaft Length	Shaft Diameter
M/ML	ERAPMLA	320mm	5mm
L/XL	ERAPXLA	320mm	10mm

## ○ Order Information

Clip	Open Applier-Standard	Endo Applier-Standard	Open Remover	Endo Remover
M	OMAPMMA	EMAPMMA	ORAPMLA	ERAPMLA
ML	OMAPMLA	EMAPMLA		
L	OMAPLLA	EMAPLLA	ORAPXLA	ERAPXLA
XL	OMAPXLA	EMAPXLA		

Clip	Open Applier-Premium	Endo Applier-Premium
ML	OPAPMLA	ENAPMLB
L	OPAPLLA	ENAPLLB
XL	OPAPXLA	ENAPXLB

### Head Office

#### Meta Biomed Co.,Ltd.

270 Osongsaengmyeong1-ro,  
Osong-eup, Heungdeok-gu,  
Cheongju-si, Chungbuk, Korea

Phone: +82 43 230 8846  
Fax: + 82 43 230 8969  
boram@meta-biomed.com  
www.meta-biomed.com

#### Meta Biomed Inc.

121 Independence Lane  
Chalfont, PA 18914, USA

Phone: +1 267 282 5893  
america@metabiomed-inc.com  
www.buymetabiomedonline.com

#### Meta Biomed Europe

Haus der Wirtschaft, Raum 3.11  
Wiesenstraße 35  
45473 Mülheim an der Ruhr, Germany

Phone: +49 208 309 9190  
europe@metabiomed-inc.com  
www.buymetabiomedonline.com